

ACCUNATE™

Home Pregnancy Test

BEFORE YOU USE THE ACCUNATE™ HOME PREGNANCY TEST

Please read this instruction leaflet completely before testing. If the foil pouch has been opened or damaged do not use. Note the expiration date on the foil pouch. Do not use if the product has expired. Store in a dry place at room temperature (below 86°F or 30°C). Do not freeze.

Keep out of reach of children.

FOR IN VITRO DIAGNOSTIC USE. NOT TO BE TAKEN INTERNALLY.

THE KIT CONTAINS

- ACCUNATE™ Home Pregnancy Test
- Product Instructions

HOW TO USE THE ACCUNATE™ HOME PREGNANCY TEST

1. Remove the test stick from the foil pouch.
2. Remove the purple cap to expose the absorbent pad.
3. Hold the test stick by the thumb grip with the absorbent pad pointing downward.
4. Place the absorbent pad into the flow of urine for at least 5 - 10 seconds. If you prefer, you may collect a urine sample in a clean, dry plastic or glass jar. Dip the absorbent pad into the urine for 5 - 10 seconds.
5. Replace the purple cap and lay the test stick flat with the result window facing up.

6. The result will appear between 1 - 3 minutes and the color will intensify over 10 minutes. DO NOT READ AFTER 10 MINUTES.

HOW TO READ THE TEST

POSITIVE RESULT (PREGNANT)


Two (2) pink bands will appear in the result window in the areas marked **C** and **T**. The color may appear as soon as one minute and will intensify over time. The degree of color may differ between the two bands.

NEGATIVE RESULT (NOT PREGNANT)

One (1) pink band will appear in the result window in the area marked **C**. This is the control band. It is an indication that sufficient urine sample has been absorbed by the "absorbent pad" and that it has flowed through the device properly to give the correct test result.

INVALID RESULT

No pink bands appear in the result window.

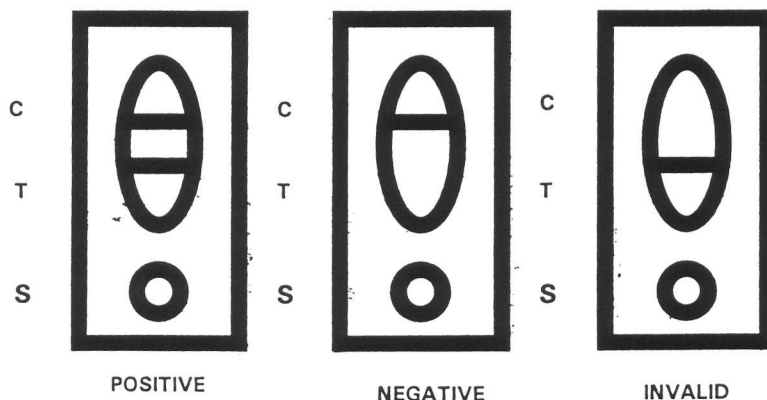

QUESTIONS AND ANSWERS ABOUT ACCUNATE™ HOME PREGNANCY TEST

How does Accunate™ Home Pregnancy Test work?

During pregnancy, the body produces a hormone, hCG (human chorionic gonadotropin). The Accunate™ Home Pregnancy Test can detect small amounts of the hormone in the urine as soon as the first day of the missed period.

When can I use the Accunate™ Home Pregnancy Test?

You can use the Accunate™ Home Pregnancy Test as soon as the first day of the missed period and at any time of day.

Can I collect the urine and test at a later time?

Urine can be collected in a clean, dry glass or plastic container and stored refrigerated for up to 24 hours. Allow the urine to come to room temperature before testing. Sometimes after refrigeration, a cloudy sediment will appear at the bottom of the container. Dip the absorbent pad of the test stick into the clear portion of the urine for 5 - 10 seconds. Replace the purple cap and lay the test stick flat with the result window facing up.

Are there drugs which can affect the test?

Any drug containing the pregnancy hormone (such as Humegon™, Pregnyl, Profasi, Pregonal, APL) can affect the result. ~ Common drugs such as painkillers, antibiotics or birth control pills will not affect the test.

Are there any other factors or medical conditions which will affect the test?

Certain medical conditions such as ovarian cysts or ectopic pregnancy can cause a false positive result. Also testing too soon after a previous pregnancy or a miscarriage (8 weeks) may also lead to a false positive result.

What do I do if the test result is negative?

If the test result is negative, you are probably not pregnant. However, if you do not get your period within the next three days, repeat the test. You may have miscalculated the due date and tested too early. If the repeated test is negative and you still do not start your period, contact your doctor since a missed period may indicate a medical condition.

What do I do if the test result is positive?

If the test is positive, it means that the Accunate™ test stick has detected hCG, the pregnancy hormone in your urine sample. You should consult your doctor to confirm pregnancy. If you are pregnant, it is important for a healthy pregnancy for both you and your baby to start a program of prenatal care.

CENOGENICS CORPORATION
Morganville, NJ 07751 U.S.A.

PG-1504-01